

# Care Stand Belts & Accessories

Medcare's Care Stand Belts are for weight-bearing patients who have more consistent and reliable motor functions. Care Stand Belts are the first and only sit-to-stand belts available with **Microban® antimicrobial product protection**. The belts are designed for use with Care Stand, Care Bar Stand, Care Stand-n-Weigh® and Care Bar Stand-n-Weigh®.




Care Stand-N-Weigh®

### Medcare Slings Contain Antimicrobial Product Protection

Microban antimicrobial protection works 24/7 for an added level of cleanliness protection. Microban inhibits the growth of microbes such as bacteria, mold and mildew that can cause stains, odors and product deterioration on our slings.



Care Bar Stand-N-Weigh®

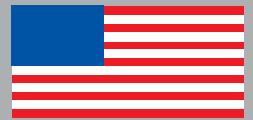

MADE IN THE USA



## Belt Sizing

Use this sizing chart as a general guide. Keep in mind that two patients that are the same weight may have a completely different body shape and size and may require different size belts.

### Belt Sizes

Belt Size	Belt Waist Size	Strap Color
Small	24" to 48"	Black
Medium	30" to 54"	Blue
Large	36" to 60"	Yellow
X-Large	42" to 66"	White

### Features

- Can be spot-cleaned or spray disinfected
- Agitation when washing will activate wash indicator signifying time for disposal



Activated wash indicator

# Care Stand Belts & Accessories (cont.)

*Med*  
HANDLE WITH CARE.



## Medcare Transport Attachment

The transport attachment is designed for use in conjunction with the stand belt. It provides additional seat and back support when needed and can also be used for added support when using the stand to ambulate.

### Item Numbers:

- 400077 – Small Transport Attachment
- 400081 – Medium Transport Attachment
- 400083 – Large Transport Attachment
- 400079 – X-Large Transport Attachment
- 400090 – Medium Belt & Attachment Combo
- 400091 – Large Belt & Attachment Combo



## Medcare Stand Belt

The Medcare stand belt is designed to go behind a patient's back and buckle around the waist. Reinforced, color-coded straps allow for easy adjustment and safety. The belt is for weight-bearing patients who have more consistent and reliable motor functions.

### Item Numbers:

- 400078 – Small Stand Belt: Fits waists 24" to 48"
- 400080 – Medium Stand Belt: Fits waists 30" to 54"
- 400082 – Large Stand Belt: Fits waists 36" to 60"
- 400084 – X-Large Stand Belt: Fits waists 42" to 66"



## Mesh Sling Bag

The mesh sling bag attaches to your Medicare Lift or Medicare Stand for convenient storage of your slings and belts. Dimensions: 16" w X 15" h



### Item Number:

- 400094 – Mesh Sling Bag

Make sure you are using slings & belts with the highest level of protection. Look for this tag.



## Sheepskin Cover for Belt

The sheepskin cover is used in conjunction with the stand belt. It provides an added level of comfort and reduces slippage, keeping the belt in place.

### Item Numbers:

- 400106 – Small Sheepskin Cover
- 400100 – Medium Sheepskin Cover
- 400102 – Large Sheepskin Cover
- 400104 – X-Large Sheepskin Cover
- 400085 – Belt & Cover Combo
- 400086 – Large Stand Belt and Sheepskin Cover Combination



## Medcare Products

151 East Cliff Road  
Burnsville, MN 55337

Toll-free: 800-695-4479

Fax: 952-894-7153

www.medcarelifts.com

© 2009 Medcare Products Inc.  
Specifications are subject to change without notice. All products referred herein are registered trademarks or trademarks of their respective holders. MICROBAN is a registered trademark of Microban Products Company.